


My DDS Lab 2.0 – Using OrthoViewer

Opening a model in OrthoViewer

Once you have installed the OrthoViewer, you can begin viewing your 3D models. (See Installing OrthoViewer for Install instructions). Start by selecting the “Open models” icon and browse to the file you wish to open. You can open both mandibular and maxillary models to view them together.

Open models Folder

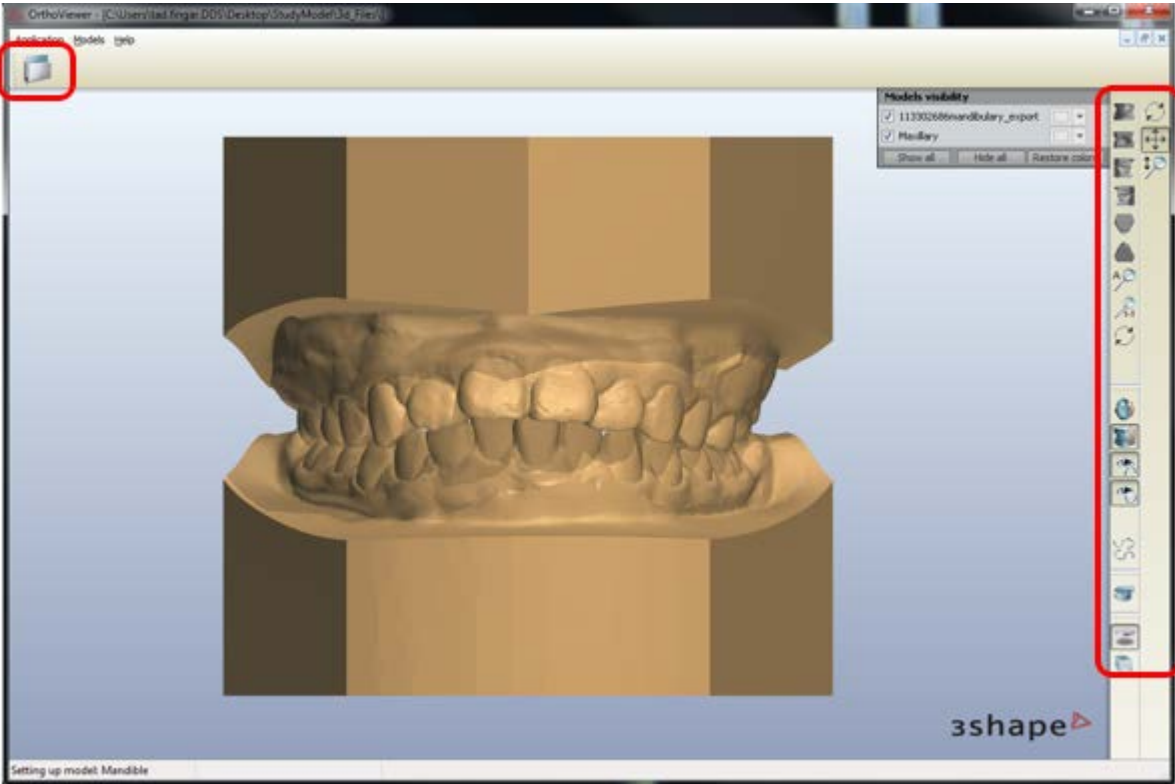

OrthoViewer Tools



OrthoViewer Tools



1) Front View

2) Rear View

3) Right View

4) Left View

5) Top View

6) Bottom View

7) Zoom All – Zooms All objects to fit on screen

8) Real Scale View – Zooms objects to a 1-1 view of the

9) Axis Locked Rotation Mode – Sets Rotation mode to Free, X, Y, or Z



10) Open Mouth View

11) Closed Mouth View

12) Show maxillary

13) Show mandibular

14) Multi View – Shows Right, Rear, Front, Left, Maxillary and Mandibular Views all at once

15) Measurements – Allows you to select two points and gives you the distance between them

16) Show Material and Shadows

17) Show as transparent – allows you to view model in transparent mode



10) Rotate View – Allows you to rotate the model. Hold and drag right mouse button to rotate

11) Pan View – Allows you to pan model. Hold and drag right mouse button to rotate

12) Zoom View – Allows you to zoom model. Hold and drag right mouse button to rotate